



HLI COUNTER LOOP INSTRUCTIONS



The design and installation of the transmission antenna in a hearing loop system is critical to proper system performance and signal reproduction. Hearing Loops LLC provides free training, technical support and design help. Please contact us for design help or approval.

ATTACH THE ANTENNA - Select a location for the LOOP pad antenna under the counter. Pick a spot that is as close to the listener's ear as possible. For best transmission, mount the pad 50% horizontal and 50% vertical. Remove the release paper from the velcro strips and mount the pad in the selected location.



External microphones can be installed through the microphone port on top of the HLI COUNTER LOOP

Secure the wire connectors from the pad into the LOOP WIRE connectors on the rear of HLI COUNTER LOOP.

ADJUST the volume by turning the volume knob on the rear of HLI COUNTER LOOP.

