



# DIGITAL CONNECTOR INSTRUCTIONS



## STEP ONE

THIS PHOTO IS FOR ILLUSTRATION PURPOSES ONLY. YOUR AUDIO SOURCE WILL PROBABLY BE DIFFERENT.



If the TV, stereo or cable box has an **OPTICAL** or **COAXIAL (COAX) DIGITAL** output, install the correct cable from the output to the **OPTICAL** or **COAX DIGITAL INPUT**. Do not use more than one input. Cable color will vary.


### OPTICAL

### COAX



## STEP TWO

Plug the cable on the **DIGITAL CONNECTOR** into the socket on the back of the **HOME loop**

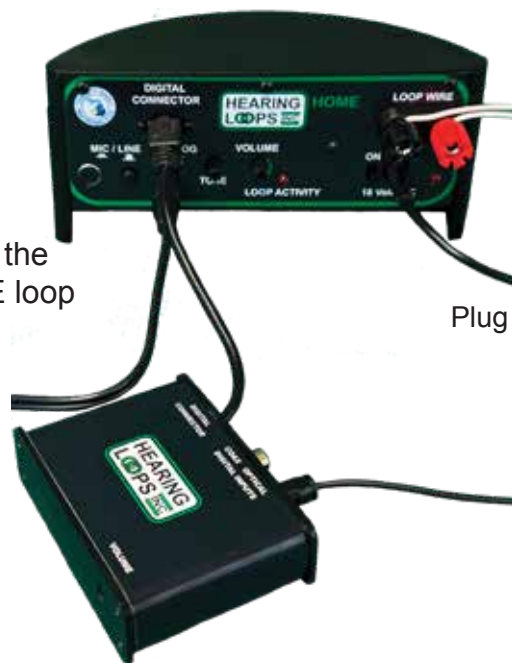

## FINAL STEP

Plug the power adapter into the wall and **HLI HOME**. Move the power switch on **HLI HOME** to the on position. Turn the **VOLUME** knob on **HLI HOME** until the **LOOP ACTIVITY** light flickers. Adjust the **TONE** as needed. The **VOLUME** control on the **DIGITAL CONNECTOR** is only needed for balance if there is an **ANALOG INPUT** in **HLI HOME** as well as the **DIGITAL CONNECTOR**.