



HLI LIGHT COMMERCIAL INSTALLATION INSTRUCTIONS



STEP 1 - Install the

wire included with HLI Light Commercial above the ceiling, around the room or under the floor. The wire must form a continuous loop around the listening area. Do not install the loop wire close to and parallel with audio lines, telephone or video cables.

Do not install the wire more than 9' above the floor. SECURE THE WIRE SAFELY. MAKE SURE IT IS NOT A TRIP HAZARD



STEP 2 - Connect the wire

Strip 1/2" of plastic from the ends of the loop wire. Loosen the red and black "LOOP WIRE" connectors. Insert one end of the bare wire into the opening in the black terminal and tighten the connector to secure the wire. Insert the other end of the wire into the opening in the red terminal and tighten the connector to secure the wire.



STEP 3 - Connect the sound source

There are a number of different types of audio outputs. Your connection depends on the model and type of TV, stereo or cable box. If your system has a left (L) and right (R) AUDIO OUT like the photo to the left, install the included cable from this output to the "ANALOG INPUT" on HLI Light Commercial.



If there is a DIGITAL COAXIAL (COAX) or OPTICAL OUT, the optional DIGITAL CONNECTOR will connect digital outputs to HLI Light Commercial.



These photos are for illustration purposes only. Your audio source will probably look different.



If there are no audio outputs, or you do not wish to purchase the Digital Connector, the sound can be picked up from a speaker with a 3.5mm microphone (not included). It is as simple as positioning the microphone in front of the speaker, plugging it into the Microphone input on the back of LC inLOOP and depressing the Microphone Power button.



STEP 4 - Adjust the volume.

Plug the power adapter into the wall and HLI Light Commercial. Turn the MIC or LINE knob until the activity light flickers. Turn the VOLUME knob until the LOOP light flickers. Adjust the TONE as needed.